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 Coloplast A/S  
3050 Denmark

23312297 v4 The Coloplast logo is a registered trademark of Coloplast A/S.  
© [2012-01] All rights reserved Coloplast A/S, 3050 Humlebæk, Denmark.

## Warning

Reuse of this single use product may create a potential risk to the user. Reprocessing, cleaning, disinfection and sterilization may compromise product characteristics which in turn create an additional risk of physical harm to or infection of the patient.

## Caution

Please seek advice from your enterostomal therapy nurse or doctor before using Convex Light products.

Coloplast accepts no liability for any injury or other loss that may arise if this product is used in a manner contrary to Coloplast's current recommendations.

## Introduction

The SenSura ostomy adhesive base plate and ostomy pouch are intended to collect output from a stoma. The SenSura adhesive base plate is intended to adhere to peristomal skin and is used for positioning of ostomy pouches.

### Adhesive - a unique double-layer adhesive

SenSura with its unique double-layer adhesive is engineered for optimal discretion and security. It protects the skin and follows all body movements without compromising on adhesion, while also keeping the skin underneath the adhesive healthy by absorbing excess moisture.

### Click coupling

The secure coupling makes a "click", confirming the coupling is securely closed.

### Filter and filter label

The air filter(s) increases the air flow and removes unpleasant odour for more discretion. Each pack is provided with filter covers. Simply stick a cover onto the opening of the filter, if you want to prevent gas from escaping or when bathing or swimming.

### Soft cover

The soft cover is made of strengthened fabric and is water repellent.

### For Convex Light only

Convex Light products are designed for those people who have an ostomy that is flush (skin level), retracted (below skin level) or ostomies located in skin folds. The appliance comes with a belt attachment on either side.

## How to use open and closed pouches

1

### 1. Cutting

Trace the size and shape of the stoma on the tracing guide. Cut the hole to fit the size and shape of the stoma. To reduce the risk of leakage and skin problems it is very important that the hole in the adhesive fits perfectly around the stoma. Cutting is easier if you use a curved pair of scissors. Ensure that the skin is clean and dry before you proceed with the application of your base plate.

2

### 2. Removal of protective film

Place a thumb on the white tab on the adhesive and pull the blue release tab away from the adhesive to remove the protective film.

3

### 3. Application of the base plate

Apply the adhesive around the stoma. To ensure a perfect fit around the stoma. To ensure a perfect fit around the stoma, press firmly against the skin starting at the bottom and smoothing upwards with your fingers.

4

### 4. Application of the pouch

Ensure the lock-ring is open. Press pouch and base plate together starting at the bottom. Make sure the pouch is securely positioned on the coupling. The pouch may be turned to optimal position.

5

### 5. Locking of the coupling

Close the lock-ring. When you hear a click the pouch is securely locked to the base plate.

6.1

### 6. Removal of pouch and base plate

#### The pouch:

1. Open the lock-ring by pressing the small tab with a finger tip.

2. Hold on to the top of the pouch. Remove the pouch from the base plate by pulling the pouch away.

#### The base plate:

3. Grab the white removal ear and pull downwards slowly.

6.2

6.3

## How to use open pouches with Hide-away/EasiClose outlet

7

### Close the outlet before use

7. Place the pouch on a table or a flat area so the coupling face is downwards. Fold the bottom part upwards and close the outlet just like an envelope.

8

8. Fold the outlet twice – until you see the velour plate.

9

9. Seal the outlet by folding the hook (Velcro) ears onto the velour plate and press firmly.

For some product version you can, if desired, fold once more as described under "Cosmetic closing of the Outlet" (point 15). These product versions aren't available in all countries.

10

**10. Opening of pouch**  
Take hold of the end of the hook (Velcro) ears. Lift both hook (Velcro) ears off the velour plate. Unfold the outlet in the position where you want to empty the pouch.

11

**11. Emptying of the pouch**  
Allow the contents to run out while you control the emptying with your fingers.

12

**12. Cleaning of the outlet**  
Use a tissue or dry wipe to clean the outlet in one sliding movement away from your body.

13

**13. Fold the small plate backwards so that only the large plate is visible. You can now clean the edge. The outlet is now cleaned and ready to be closed again.**

14

**14. Re-closing of the outlet**  
Bend the lower small plate over the upper large plate. Then fold the outlet twice – until you see the velour plate. Seal the outlet by folding the hook (Velcro) ears onto the velour plate and press firmly.

15

**15. Cosmetic closing of the outlet (only for some product versions)**  
The whole closing system can be folded up under the soft cover and attached to the Velcro dot. The pouch will now become more discreet.

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